

TECHNICAL DATA SHEET

Neitum Physiological Saline **Sterile nasal and ophthalmic solution in single-dose container**

Indications :

Physiological saline solution is recommended for washing the nasal cavities and for daily eye cleansing, for hygiene of the eyes and nose in infants, children, and adults.

- **Nasal use:** Physiological saline helps clean the noses of very young children, which are often blocked by mucus. It promotes the elimination of excess secretions. Thanks to its moisturizing action, it quickly relieves nasal dryness in adults.
- **Ophthalmic use:** Physiological saline can be used for eye hygiene in infants by instillation or by applying impregnated compresses or cotton pads.

Physiological saline can also be used for eye hygiene in children and adults by washing, instillation, eye baths, or applying impregnated compresses or cotton pads.

Form and Presentation :

Box of 20 single-dose containers of 5 mL.

Composition :

Sodium Chloride 0.9 g

Purified water q.s. to 100 mL

Notes :

Physiological saline can be used as often as needed, without risk over long periods.

If using a nasal or ophthalmic medication, physiological saline should be applied *before* the medication and not afterward.

Precautions for use :

- Single use only. Do not inject.
- Keep out of reach of children.
- Do not use after the expiration date.
- Do not use the same single-dose container for both nose and eyes.
- Store at room temperature.

Any opened container must not be reused, as the product contains no preservatives. Because this medical device contains no preservatives, reuse may result in administering a contaminated solution.

Instructions for use:

Detach a single-dose container and open it by twisting the top part.

Nasal use: Insert the tip of the single-dose container into the nostril with the head slightly tilted and gently squeeze the container. Then blow the nose or use a nasal aspirator.

For infants, instill the product with minimal pressure to avoid any risk of middle-ear contamination.

Ophthalmic use: Moisten a cotton pad and gently wipe the baby's eyelids, or pour a few drops into the eye and wipe with a sterile compress.

For adults, the product may be used for instillation, washing, or eye baths.

The CE marking certification was obtained on 25/08/2000.

Date of revision of the instructions for use: November 2012.

Manufacturer :

Laboratoire UNITHER
151, Rue André Durouchez
Espace Industriel Nord
CS 28028 - 80084 AMIENS Cedex 2
FRANCE

Marketing Authorization Holder :

Laboratoires FRILAB
104 Blvd Auguste Blanqui
75013 PARIS – FRANCE

Distributor :

Laboratoires FRILAB SA
17, rue des Pierres du Niton
1207 Geneva – SWITZERLAND